**CE** Instructions for the Use of  
global<sup>®</sup> w/ HEPES

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(Références : LGGH-050, LGGH-100, LGGH-500)

**Precautions and Warnings**

- Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician (or properly licensed practitioner).
- Caution:** The user should read and understand the Instructions for Use, Precautions and Warnings, and be trained in the correct procedure before using the global<sup>®</sup> w/ HEPES for embryo washing and handling.
- Not to be used for injection.
- Do not use the product if:
  - the product packaging appears damaged or if the seal is broken
  - the expiry date has been exceeded
  - the product becomes discolored, cloudy, or shows evidence of particulate matter
- global<sup>®</sup> w/ HEPES contains only a low concentration of sodium bicarbonate and should not be gassed, or used for culture under CO<sub>2</sub>.
- global<sup>®</sup> w/ HEPES contains the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.
- To avoid problems with contamination, practice aseptic techniques.
- Discard unused medium within 7 days of opening. Do not use after expiry date.

**General Information**

**Indications for Use**

global<sup>®</sup> w/ HEPES is designed to be used for embryo washing and handling.

**Storage and Shelf Life**

Store at 2-8°C and protected from light. Ten (10) weeks from the date of manufacture.

**Composition**

A HEPES-buffered medium is required when embryos are handled outside of a CO<sub>2</sub> incubator.

Sodium Chloride, Sodium Bicarbonate, L-Alanine, L-Glutamic Acid, L-Lysine HCl, L-Threonine, Phenol Red

Potassium Chloride, Glucose, L-Arginine HCl, Glycyl-Glutamine, L-Methionine, L-Tryptophan, HEPES

Calcium Chloride, Lactate Na Salt, L-Asparagine, L-Histidine, L-Phenylalanine, L-Tyrosine, Gentamicin Sulfate\* (10 µg/ml)

Potassium Phosphate, Potassium Phosphate, L-Aspartic Acid, L-Isoleucine, L-Proline, L-Valine

Magnesium Sulfate, Glycine, L-Cystine, L-Leucine, L-Serine, EDTA

\*from therapeutic-grade source material

**Quality Control Specifications**

Media	Specification
pH	7.2-7.4
Osmolality – (mOsm)	260-270
LAL Endotoxin	≤ 0.5 EU/ml
Sterility Test, membrane filtration	Negative
1-cell Mouse Embryo Assay (% blastocysts at 96h of culture after 1 h exposure)	≥ 80%

**Instructions for Use**

The procedures described below have been found to be effective for the preparation of global<sup>®</sup> w/ HEPES for embryo washing and handling.

- Open the bottle and add a sufficient volume of protein (Human Serum Albumin authorized for EU Distribution) to produce a desired final concentration of protein in the medium. Cap and gently agitate the bottle to completely mix the protein into the medium.
- Aliquot the volume of supplemented global<sup>®</sup> w/ HEPES needed for one patient or one day into appropriate embryo-safe culture tubes. Cap the tubes tightly and store at 2-8°C, protected from light.
- To wash oil, place 50 ml of global w/ HEPES (without protein) into a tissue culture flask and overlay with 250 ml of oil. Agitate flask. Store at 2-8°C.
- The night before intended use aliquot a sufficient volume of washed oil into embryo-safe culture tubes. Cap the tubes and place them into a heating block at 37°C, overnight.
- The night before or the morning of intended use, place a sufficient number of tubes of supplemented global<sup>®</sup> w/ HEPES into a heating block at 37°C.
- Prepare a sufficient number of plates containing micro-drops of supplemented global<sup>®</sup> w/ HEPES under washed oil.
- Cover and seal the dishes and place into the incubator for final temperature equilibration for 1-2 hours.

**Symbols**

<b>STERILE A</b>	<b>RX Only</b>	<b>REF</b>	<b>LOT</b>
Sterilize by Irradiation	By Prescription Only	Catalogue Number	Batch Code

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